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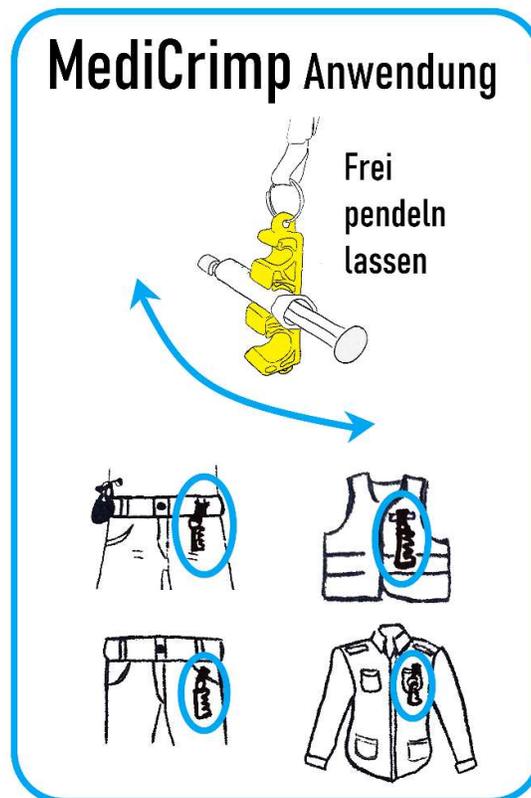
www.medicrimp.ch
kontakt@medicrimp.ch

MediCrimp© English

- Increases patient safety, the risk of medication mix-up is minimized.
- It is significantly easier to keep an overview.
- Medication is directly accessible, no more tedious searching.
- Easy exchange of syringes with team partner (no need to put down the syringe anymore)
- Robust design
- Flexible options to attach syringes
- Compatible with the most popular brands of syringes
- Clothing-friendly clip

From conception to production, MediCrimp is 100% made in Switzerland.

Holds syringes on gilet, belt, pocket, pen tab, jacket, etc.

USE

The use of the MediCrimp is only permitted if the user has read and understood the following information:

- **The MediCrimp must be able to HANG FREELY** to avoid knocking off the syringes. Attachment with clip and ring is recommended.
- In order to reduce the risk of the syringe being knocked off, the use of combi-stoppers on the syringes is recommended. Using syringes with recapped needles is not recommended.
- If the MediCrimp is attached to the side (flap on the back), it is ideally attached to the shirt or jacket. It is essential to ensure that it dangles freely.
- The attachment clip must always be attached in the direction of pull of the MediCrimp.
- Make sure that the syringes are clipped in correctly. Clipping is also possible with the ampoule on the syringe (adhesive tape).
- Always use the matching slot for the corresponding syringe size. Clip 2ml syringes into the smallest, 5ml syringes into the middle and 10ml syringes into the largest slot.

Tested with Braun and BD syringes in the sizes 10ml, 5ml and 2ml.

The application of the MediCrimp © is the responsibility of the user. In the event of improper handling, loss or theft of a syringe containing medication, or use of a medication by unqualified persons, no liability is accepted. In any case, the user is responsible for handling the medication and for keeping it safe.

Krimper.ch assumes responsibility for manufacturing and material defects.

www.medicrimp.ch

kontakt@medicrimp.ch